

**BESA<sup>®</sup>**

**Epilepsy 2.0**

**How to hide events in the event list**

August 2016


Please start [EEG Review](#) or [Seizure Review](#). The event list at the right shows all available events which have been read from the file or have been added by previous reviewing sessions.

Evt	Rep		Capt EEG	Capt Map
SzD	Usr2	Usr3	Usr4	Hide
<b>0</b>				
Type	Description	Time	Rep	Map
Seg	File 0 Start	07:25:58.738	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	Montage:Doppelte Banane (MON), Ref	07:25:58.812	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	Recording Analyzer - XLSpike	07:25:58.941	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
Cmt	Video Recording ON	07:25:59.511	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
Cmt	XLSpike	07:58:20.851	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	XLSpike	08:29:06.714	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	XLSpike	08:34:27.511	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	XLSpike	08:36:25.414	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	XLSpike	08:36:55.226	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	XLSpike	08:37:49.011	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	XLSpike	08:38:18.253	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	XLSpike	08:38:40.519	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	XLSpike	08:41:49.164	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	XLSpike	08:43:12.011	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	XLSpike	08:43:56.171	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	XLSpike	08:44:36.804	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	XLSpike	08:45:51.941	<input type="checkbox"/>	<input type="checkbox"/>

In this example the event list shows many **XLSpike** events which have been added by the XLTEK system. We are now going to hide the **XLSpike** events for all further reviews. Finally this document explains how to make them visible immediately.

Please right click the event, which you want to hide permanently for all further reviews.

Evt		Rep	Capt EEG	Capt Map
SzD	Usr2	Usr3	Usr4	Hide
Type	Description	Time	Rep	Map
Seg	File 0 Start	07:25:58.738	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	Montage:Doppelte Banane (MON), Ref	07:25:58.812	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	Recording Analyzer - XLSpike	07:25:58.941	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
Cmt	Video Recording ON	07:25:59.511	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
Cmt	XLSpike	07:58:20.851	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	XLSpike	08:29:06.714	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	XLSpike	08:34:27.511	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	XLSpike	08:36:25.414	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	XLSpike	08:36:55.226	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	XLSpike	08:37:49.011	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	XLSpike	08:38:18.253	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	XLSpike	08:38:40.519	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	XLSpike	08:41:49.164	<input type="checkbox"/>	<input type="checkbox"/>

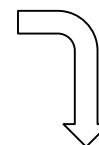

Evt		Rep	Capt EEG	Capt Map
SzD	Usr2	Usr3	Usr4	Hide
Type	Description	Time	Rep	Map
Seg	File 0 Start	07:25:58.738	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	Montage:Doppelte Banane (MON), Ref	07:25:58.812	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	Recording Analyzer - XLSpike	07:25:58.941	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
Cmt	Video Recording ON	07:25:59.511	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
Seg	File 1 Start	09:32:06.312	<input type="checkbox"/>	<input type="checkbox"/>
Seg	File 1 Segment	09:42:37.292	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	Bookmark	10:36:42.484	<input type="checkbox"/>	<input type="checkbox"/>
Seg	File 4 Start	10:37:44.718	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
Img	Av33 / Filters: 2 - 35 Hz Notch 50 Hz	11:09:51.015	<input type="checkbox"/>	<input type="checkbox"/>
Seg	File 2 Start	11:09:57.492	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
Seg	File 5 Start	11:10:01.312	<input type="checkbox"/>	<input type="checkbox"/>
Seg	File 5 Segment	13:00:08.070	<input type="checkbox"/>	<input type="checkbox"/>
Seg	File 6 Start	13:18:20.531	<input type="checkbox"/>	<input type="checkbox"/>
Seg	File 7 Start	15:23:28.343	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	Video Recording OFF	15:46:19.125	<input type="checkbox"/>	<input type="checkbox"/>

After selecting the **Hide** menu item, all events of this type are hidden. The event list is updated and the **Hide** button retrieves the pushed state.

**Hide**

If you now start a review for a different dataset or patient, the events **Cmt XLSpike** are hidden by default. Press the **Hide** button to make the events temporary visible for the current review. Press the **Hide** button again to hide the events.



If you right-click the **Hide** button a menu appears. It shows a list of hidden events.

Evt	Rep		Capt EEG	Capt Map
SzD	Usr2	Usr3	Usr4	Hide
				<b>Cmt XLSpike</b>
				Clear Hidden Event List
Type	Description			
Seg	File 0 Start			
Cmt	Montage:Doppelte Banane (MON), Ref	07:25:58.812	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	Recording Analyzer - XLSpike	07:25:58.941	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
Cmt	Video Recording ON	07:25:59.511	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
Seg	File 1 Start	09:32:06.312	<input type="checkbox"/>	<input type="checkbox"/>
Seg	File 1 Segment	09:42:37.292	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	Bookmark	10:36:42.484	<input type="checkbox"/>	<input type="checkbox"/>
Seg	File 4 Start	10:37:44.718	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
Img	Av33 / Filters: 2 - 35 Hz Notch 50 Hz	11:09:51.015	<input type="checkbox"/>	<input type="checkbox"/>
Seg	File 2 Start	11:09:57.492	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
Seg	File 5 Start	11:10:01.312	<input type="checkbox"/>	<input type="checkbox"/>
Seg	File 5 Segment	13:00:08.070	<input type="checkbox"/>	<input type="checkbox"/>
Seg	File 6 Start	13:18:20.531	<input type="checkbox"/>	<input type="checkbox"/>
Seg	File 7 Start	15:23:28.343	<input type="checkbox"/>	<input type="checkbox"/>
Cmt	Video Recording OFF	15:46:19.125	<input type="checkbox"/>	<input type="checkbox"/>

If you select the **Clear Hidden Event List** menu item, the list of hidden events is cleared and all permanently hidden events become visible again.