BESA MRI 3.0 - Instructions for Use		 BESA®		
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93924	003/07.2022	Kornwebel, Michael 2022-07-05	Scherg, Tobias 2022-07-05	Kornwebel, Michael 2022-07-05

Description:

Please refer to the country code related to your preferred language.

(Alpha-2 code country codes used are listed in <https://www.iso.org/obp>).

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D-50679 Köln

Valid for all languages except DE and EN.

AU, CA, GB, IE, IN, MT, MY, NZ, US, ZA: This document contains the instructions for use and safety instructions, as well as technical description for this BESA MRI 3.0.

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BESA MRI 3.0 - Instructions for Use				
Document ID 93924	Revision Number 003/07.2022	Edited by Kornweibel, Michael 2022-07-05	Reviewed by: Scherg, Tobias 2022-07-05	Released by / Date Kornweibel, Michael 2022-07-05

[Table of Content](#)

AU, CA, GB, IE, IN, MT, MY, NZ, US, ZA.....	3
DE, AT, CH, LI, LU.....	8
AE, QA, SA, SA, UA	13
AR, CL, CO, ES, MX.....	17
CN, HK, TW	22
CY, TR	27
IT.....	32
JP.....	37
KR.....	42
NL, BE.....	47
PL.....	52
PT, BR	57
SE	62
TH	67

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA MRI 3.0 - Instructions for Use		 BESA®		
Document ID 93924	Revision Number 003/07.2022	Edited by Kornweibel, Michael 2022-07-05	Reviewed by: Scherg, Tobias 2022-07-05	Released by / Date Kornweibel, Michael 2022-07-05

AU, CA, GB, IE, IN, MT, MY, NZ, US, ZA

Instructions for Use

The referenced term “BESA MRI 3.0” includes all sub-versions of the product.

This document together with the following documents form the complete Instructions for Use for this BESA product:

- BESA MRI 3.0 - IntendedUse+ProductClassification
- BESA MRI 3.0 - User Manual
- BESA MRI 3.0 - Update History
- BESA License Agreement
- BESA Product Symbols - Meaning
- BESA License Key – Instructions for Use (and within referred documents)

Current and previous versions of documents are available on BESA website:

<https://www.besa.de/downloads/user-documentation/>

It is mandatory to read the complete Instructions for Use documents, before using the product.

At any time during the lifetime of this product, you can request the above-mentioned documents in printed paper form at no additional cost.

Product Name

BESA MRI 3.0.0.0

This BESA product is a medical device.

Product UDI

UDI-DI: (1)4262364230011

UDI-PI: (8012)03000000

Manufacturer

BESA GmbH
 Freihamer Strasse 18
 82166 Graefelfing/Germany

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA MRI 3.0 - Instructions for Use				
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93924	003/07.2022	Kornweibel, Michael 2022-07-05	Scherg, Tobias 2022-07-05	Kornweibel, Michael 2022-07-05

Symbols used for Product

This product is marked with a sign, additional symbols are described in document: "BESA Product Symbols – Meaning", available on the BESA website.

Representative inside the European Union

Yes No

Life cycle

The life cycle of the product ends on 2024-05-18.

Incident

In case of a serious incident or near-incident caused by BESA software, the licensee should directly report to incident@besa.de or follow the guidelines stipulated on the BESA website. (<https://www.besa.de/contact/incident/>)

Safety Instructions and Technical Description

System Requirements

This BESA product is a stand-alone SW product compatible with personal computers running under the following hardware/OS:

- Operating system: Windows® 10 Touch not supported
 Windows® 7 Touch not supported
- CPU: minimum 2 GHz
- RAM: minimum 8 GB
- Display resolution: minimum 1280 × 1024 pixels
- Graphics card: OpenGL 2.0 with 16 MB RAM or more

This BESA product was tested on the following operating system configurations:

- Windows 10 Professional 64bit version 2004
- Windows 10 Professional 64bit version 20H2
- Windows 7 Professional 64bit Service Pack 1

Please note that changes of the hardware or software platforms (e.g., with patches/updates of antivirus/firewall software, system libraries, firmware, and others) and/or changes to the IT network may result in the possibility of safety and/or security vulnerabilities, and/or reduced functionality of BESA products. It is recommended to use the system combinations as defined within this document.

Please also note that the use of BESA products in an IT-network could result in previously unidentified risks to patients, users or third parties, and that the user is responsible to identify, analyze, evaluate, and control

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA MRI 3.0 - Instructions for Use		 BESA®		
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93924	003/07.2022	Kornweibel, Michael 2022-07-05	Scherg, Tobias 2022-07-05	Kornweibel, Michael 2022-07-05

these risks. Any subsequent changes to the IT-network could introduce new risks and require additional analysis.

The BESA License Key is an essential accessory product to BESA products, and shall either be subject to constant supervision, or shall be connected to machines located in areas with access restriction. Failure of the IT-network and/or operating system on which the BESA License Key is used, or failure of the BESA License Key in general, may render BESA products which use the BESA License Key as an accessory product unusable.

Data Protection

BESA applications offer the possibility to save files that may include „personal data“ as specified in GDPR (Directive 95 /46 EC, General Data Protection Regulation). These files may also include protected health information. Users are strongly advised to inform themselves about applicable regulations. With respect to BESA applications, we especially recommend to not use patient names as part of file names (e.g. for exported data or MRI projects), or to store any data that includes personal data or protected health information on a shared network drive, to which multiple users may have access.

Use in combination with other products

BESA MRI 3.0 is designed to be used in combination with the following medical device software:

- BESA Research 7.1

BESA releases are constantly checked with other installations (e.g. MS Office etc.) on the same hardware or software platforms as defined in section System Requirements.

If any incompatibilities are observed, they are documented within the Update History of the product. For further questions and information please use the channels provided on www.besa.de.

Application Environment

Please Note: It is the User’s responsibility to ensure the safety of combined medical or medical-&-non-medical-devices particularly installed in the patient environment (whether in institutes, clinics or practitioners’ location).

The user of our product has to ensure that such combination fulfil the same safety condition as a single medical device according to IEC 60601-1.

Such combination with any non-medical device establishes are “in-house medical system” – observe the latest Medical Device Operator Ordinance (e.g. MedBetreiberVerordnung, Germany 2020) or your national Operator Laws.

We recommend you to document / retrace our Software version/updates installed in your facility / network in a MPG-Medizingeräte-Buch (medical facilities book), similar to those needed for Hardware.

This BESA product does not affect the possibility of using any medical computer within patient environment.

BESA GMBH is not liable for any use/installation beyond the defined PC network.

Installation and Un-Installation

The installation and un-installation of the product must be carried out by the system administrator or an authorized person. This staff bears full responsibility for correct setup of the product within the network.

This BESA product shall only be installed on PC with password-protected user accounts.

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA MRI 3.0 - Instructions for Use		 BESA®		
Document ID 93924	Revision Number 003/07.2022	Edited by Kornweibel, Michael 2022-07-05	Reviewed by: Scherg, Tobias 2022-07-05	Released by / Date Kornweibel, Michael 2022-07-05

Product updates must also be carried out by the system administrator or an authorized person.
Please note, that in case of any problems during the installation, the setup.exe will trigger a rollback of current changes. In that case, please figure out and eliminate the issue, and run the setup.exe again.

To install the product, please follow these steps:

1. Please download the BESA product from our website:
<https://www.besa.de/downloads/besa-mri/>
2. Run the downloaded setup.exe
3. Please follow the on-screen instructions
4. Towards the end of the installation process, the installation of the BESA License Key setup is triggered, please follow the on-screen instructions, and please refer to “BESA License Key – Instructions for Use” for further information.

Once the installation is complete, proceed with the initial setup as explained in the User Manual.

Exclusions for Use

In countries where the CE certification for medical devices is not recognized, it is NOT ALLOWED to use this software directly or indirectly for medical purposes and / or treatment of humans.
Not for diagnostic use in the United States of America!

Contraindication to patients

Not applicable.

BESA is not liable for the use outside of the rules defined in the Instructions for Use documents.

Manufacture, Transport, Storage and Usage conditions

Environmental conditions for manufacturing

N/A

Environmental conditions for Transport, Storage and Usage

N/A

Countries and Markets

This BESA product may be sold in the following countries and markets:

Austria	Germany	Malta	Spain	China	Russia
Belgium	Greece	Macedonia	Sweden	Colombia	Saudi Arabia
Bulgaria	Hungary	Netherlands	Switzerland	India	Singapore
Croatia	Iceland	Norway	Turkey	Iran	South Africa

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA MRI 3.0 - Instructions for Use				
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93924	003/07.2022	Kornwebel, Michael 2022-07-05	Scherg, Tobias 2022-07-05	Kornwebel, Michael 2022-07-05

Cyprus	Ireland	Poland	United Kingdom	Israel	South Korea
Czech Republic	Italy	Portugal	Australia	Japan	Taiwan
Denmark	Latvia	Romania	Argentina	Malaysia	Thailand
Estonia	Liechtenstein	Serbia	Brazil	Mexico	Turkmenistan
Finland	Lithuania	Slovakia	Canada	New Zealand	UAE
France	Luxembourg	Slovenia	Chile	Qatar	USA

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA MRI 3.0 - Instructions for Use		 BESA®		
Document ID 93924	Revision Number 003/07.2022	Edited by Kornweibel, Michael 2022-07-05	Reviewed by: Scherg, Tobias 2022-07-05	Released by / Date Kornweibel, Michael 2022-07-05

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