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BESA Research 7.1 - Instructions for Use				
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93926	003/07.2022	Kornwebel, Michael 2022-07-05	Scherg, Tobias 2022-07-05	Scherg, Tobias 2022-07-05

[Table of Content](#)

AU, CA, GB, IE, IN, MT, MY, NZ, US, ZA.....3

DE, AT, CH, LI, LU.....8

AE, QA, SA, UA13

AR, CL, CO, ES, MX.....18

CN, HK, TW23

CY, TR28

IT33

JP38

KR43

NL, BE.....48

PL.....53

PT, BR58

SE63

TH68

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA Research 7.1 - Instructions for Use		 BESA®		
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93926	003/07.2022	Kornweibel, Michael 2022-07-05	Scherg, Tobias 2022-07-05	Scherg, Tobias 2022-07-05

AU, CA, GB, IE, IN, MT, MY, NZ, US, ZA

Instructions for Use

The referenced term “BESA Research 7.1” includes all sub-versions of the product.

This document together with the following documents form the complete Instructions for Use for this BESA product:

- BESA Research 7.1 - IntendedUse+ProductClassification
- BESA Research 7.1 – Getting Started
- BESA Research 7.1- Tutorial
- BESA Research 7.1- Update History
- BESA License Agreement
- BESA Product Symbols - Meaning
- BESA License Key – Instructions for Use (and within referred documents)

Current and previous versions of documents are available on BESA website:

<https://www.besa.de/downloads/user-documentation/>

It is mandatory to read the complete Instructions for Use documents, before using the product.

At any time during the lifetime of this product, you can request the above-mentioned documents in printed paper form at no additional cost.

Product Name

BESA Research 7.1.2.1

This BESA product is a medical device.

Product UDI

UDI-DI: (1)4262364230004

UDI-PI: (8012)07010201

Manufacturer

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA Research 7.1 - Instructions for Use		 BESA®		
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93926	003/07.2022	Kornweibel, Michael 2022-07-05	Scherg, Tobias 2022-07-05	Scherg, Tobias 2022-07-05

BESA GmbH
 Freihamer Strasse 18
 82166 Graefelfing/Germany

Symbols used for Product

This product is marked with a sign, additional symbols are described in document: "BESA Product Symbols – Meaning", available on the BESA website.

Representative inside the European Union

Yes No

Life cycle

The life cycle of the product ends on 2024-05-18.

Incident

In case of a serious incident or near-incident caused by BESA software, the licensee should directly report to incident@besa.de or follow the guidelines stipulated on the BESA website. (<https://www.besa.de/contact/incident/>)

Safety Instructions and Technical Description

System Requirements

This BESA product is a stand-alone SW product compatible with personal computers running under the following hardware/OS:

- Operating system: Windows® 10 Touch not supported
 Windows® 7 Touch not supported
- CPU: minimum 2 GHz
- RAM: minimum 8 GB
- Display resolution: minimum 1280x1024 pixels
- Graphics card: OpenGL 2.0 with 16 MB RAM or more

This BESA product was tested on the following operating system configurations:

- Windows 10 Professional 64bit version 2004
- Windows 10 Professional 64bit version 20H2
- Windows 7 Professional 64bit Service Pack 1

Please note that changes of the hardware or software platforms (e.g., with patches/updates of antivirus/firewall software, system libraries, firmware, and others) and/or changes to the IT network may result in the possibility

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA Research 7.1 - Instructions for Use				
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93926	003/07.2022	Kornweibel, Michael 2022-07-05	Scherg, Tobias 2022-07-05	Scherg, Tobias 2022-07-05

of safety and/or security vulnerabilities, and/or reduced functionality of BESA products. It is recommended to use the system combinations as defined within this document.

Please also note that the use of BESA products in an IT-network could result in previously unidentified risks to patients, users or third parties, and that the user is responsible to identify, analyze, evaluate, and control these risks. Any subsequent changes to the IT-network could introduce new risks and require additional analysis.

The BESA License Key is an essential accessory product to BESA products, and shall either be subject to constant supervision, or shall be connected to machines located in areas with access restriction. Failure of the IT-network and/or operating system on which the BESA License Key is used, or failure of the BESA License Key in general, may render BESA products which use the BESA License Key as an accessory product unusable.

Data Protection

BESA applications offer the possibility to save files that may include „personal data“ as specified in GDPR (Directive 95 /46 EC, General Data Protection Regulation). These files may also include protected health information. Users are strongly advised to inform themselves about applicable regulations. With respect to BESA applications, we especially recommend to not use patient names as part of file names (e.g. for exported data or MRI projects), or to store any data that includes personal data or protected health information on a shared network drive, to which multiple users may have access.

Use in combination with other products

BESA Research 7.1 is designed to be used in combination with the following medical device software:

- BESA MRI 3.0
- BESA Connectivity 1.0

BESA Research 7.1 is designed to be used in combination with the following non-medical software:

- BESA Statistics 2.1
- Matlab Version: any major release between 2006a and 2019b

BESA releases are constantly checked with other installations (e.g. MS Office etc.) on the same hardware or software platforms as defined in section System Requirements.

If any incompatibilities are observed, they are documented within the Update History of the product. For further questions and information please use the channels provided on www.besa.de

Application Environment

Please Note: It is the User’s responsibility to ensure the safety of combined medical or medical-&-non-medical-devices particularly installed in the patient environment (whether in institutes, clinics or practitioners’ location).

The user of our product has to ensure that such combination fulfil the same safety condition as a single medical device according to IEC 60601-1.

Such combination with any non-medical device establishes are “in-house medical system” – observe the latest Medical Device Operator Ordinance (e.g. MedBetreiberVerordnung, Germany 2020) or your national Operator Laws.

We recommend you to document / retrace our Software version/updates installed in your facility / network in a MPG-Medizingeräte-Buch (medical facilities book), similar to those needed for hardware.

This BESA product does not affect the possibility of using any medical computer within patient environment.

BESA GmbH is not liable for any use/installation beyond the defined PC network.

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA Research 7.1 - Instructions for Use				
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93926	003/07.2022	Kornwebel, Michael 2022-07-05	Scherg, Tobias 2022-07-05	Scherg, Tobias 2022-07-05

Installation and Un-Installation

The installation and un-installation of the product must be carried out by the system administrator or an authorized person. This staff bears full responsibility for correct setup of the product within the network.

This BESA product shall only be installed on PC with password-protected user accounts.

Product updates must also be carried out by the system administrator or an authorized person.

Please note, that in case of any problems during the installation, the setup.exe will trigger a rollback of current changes. In that case, please figure out and eliminate the issue, and run the setup.exe again.

To install the product, please follow these steps:

1. Please download the BESA product from our website:
<https://www.besa.de/downloads/besa-research/besa-research-7-1/>
2. Run the downloaded setup.exe
3. Please follow the on-screen instructions
4. Towards the end of the installation process, the installation of the BESA License Key setup is triggered, please follow the on-screen instructions, and please refer to “BESA License Key – Instructions for Use” for further information.
5. Towards the end of the installation process, a dialog window opens to perform the setup of the MATLAB interface. Please follow the on-screen instructions.

Once the installation is complete, proceed with the initial setup as explained in “BESA Research 7.1 - Getting Started”.

Exclusions for Use

In countries where the CE certification for medical devices is not recognized, it is NOT ALLOWED to use this software directly or indirectly for medical purposes and / or treatment of humans.

Not for diagnostic use in the United States of America!

Contraindication to patients

N/A

BESA is not liable for the use outside of the rules defined in the instructions for use documents.

Manufacture, Transport, Storage and Usage conditions

Environmental conditions for manufacturing

N/A

Environmental conditions for Transport, Storage and Usage

N/A

Countries and Markets

This BESA product may be sold in the following countries and markets:

Austria Germany Malta Spain China Russia

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA Research 7.1 - Instructions for Use				
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93926	003/07.2022	Kornwebel, Michael 2022-07-05	Scherg, Tobias 2022-07-05	Scherg, Tobias 2022-07-05

Belgium	Greece	Macedonia	Sweden	Colombia	Saudi Arabia
Bulgaria	Hungary	Netherlands	Switzerland	India	Singapore
Croatia	Iceland	Norway	Turkey	Iran	South Africa
Cyprus	Ireland	Poland	United Kingdom	Israel	South Korea
Czech Republic	Italy	Portugal	Australia	Japan	Taiwan
Denmark	Latvia	Romania	Argentina	Malaysia	Thailand
Estonia	Liechtenstein	Serbia	Brazil	Mexico	Turkmenistan
Finland	Lithuania	Slovakia	Canada	New Zealand	UAE
France	Luxembourg	Slovenia	Chile	Qatar	USA

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA Research 7.1 - Instructions for Use		 BESA®		
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93926	003/07.2022	Kornweibel, Michael 2022-07-05	Scherg, Tobias 2022-07-05	Scherg, Tobias 2022-07-05

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