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BESA License Key - Instructions for Use		 BESA®		
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93944	004/07.2022	Kornwebel, Olga 2022-07-05	Kornwebel, Michael 2022-07-05	Scherg, Tobias 2022-07-05

[Table of Content](#)

AU, CA, GB, IE, IN, MT, MY, NZ, US, ZA.....	3
DE, AT, CH, LI, LU.....	14
AE, QA, SA, UA	26
AR, CL, CO, ES, MX.....	37
CN, HK, TW	50
CY, TR	62
IT	74
JP.....	86
KR.....	98
NL, BE.....	111
PL.....	123
PT, BR	135
SE	147
TH	159

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA License Key - Instructions for Use			 BESA®	
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93944	004/07.2022	Kornwebel, Olga 2022-07-05	Kornwebel, Michael 2022-07-05	Scherg, Tobias 2022-07-05

AU, CA, GB, IE, IN, MT, MY, NZ, US, ZA

Instructions for Use

The referenced term “BESA License Key 2.0” includes all subversions of the product.

This document, and following documents form the complete Instructions for Use for this BESA product:

BESA License Key 2.0 - IntendedUse+ProductClassification

BESA License Key 2.0 - Update History

BESA License Agreement

BESA Product Symbols - Meaning

Current and previous versions of documents are available on BESA website:

<https://www.besa.de/downloads/user-documentation/>

It is mandatory to read the complete Instructions for Use documents, before using the product.

At any time during the lifetime of this product, you can request the above-mentioned documents in printed paper form at no additional cost.

Product Name

BESA License Key 2.0 is an accessory product to BESA medical devices.

Product Picture

Product UDI

UDI-DI: (1)426236423003

UDI-PI: NOTE: the UDI-PI is individually for each License Key 2.0, including the serial number. It is available on the label applied to BESA Box.

Manufacturer

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA License Key - Instructions for Use				
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93944	004/07.2022	Kornwebel, Olga 2022-07-05	Kornwebel, Michael 2022-07-05	Scherg, Tobias 2022-07-05

BESA GmbH
Freihamer Strasse 18
82166 Graefelfing/Germany

NOTE: With the use of the symbol on the BESA License Key 2.0 itself, BESA does not claim manufactureship, but rather provides a contact address.

Symbols used for Product

This product is marked with a CE sign, additional symbols are described in document: "BESA Product Symbols - Meaning", available on BESA website.

Representative inside the European Union

☐ Yes ☒ No

Life cycle

The life cycle of all dongles subsumed under BESA License Key 2.0 ends on 2031-04-01.

Note:

The manufacturer of the BESA License Key 2.0 defines the battery life / warranty as 4 years after production (see "Sentinel HL Data Sheet.pdf", roXtra ID 83890, revision 001/06.2018). In document "011-010189-001_Sentinel HL_Rev_J-signed.pdf" (roXtra ID 82774, revision 001/10.2019), the China RoHS mark indicates an Environmental Protection Use Period (EPUP) of 10 years. There is no further information on the life cycle of the BESA License Key 2.0 in the accompanying documents supplied by the manufacturer.

In compliance with the statements of the manufacturer, BESA acknowledges the life cycle of the BESA License Key 2.0 as 10 years.

(Note: even if a battery has run out of power and is not functioning, the BESA License Key 2.0 will still provide licenses for BESA products. The battery is only required if at least one license on the BESA License Key 2.0 is a time-limited license. This scenario may apply if there is a transition phase where a BESA product has been ordered and delivered but payment has not yet been received by BESA. Once payment has been received, a perpetual license is provided to the customer).

BESA takes care only to supply BESA License Keys with a life cycle which is at least as long as the life cycle of the accompanying BESA products.

Incident

In case of a serious incident or near-incident caused by BESA software, the licensee should directly report to incident@besa.de or follow the guidelines stipulated on the BESA website. (<https://www.besa.de/contact/incident/>)

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA License Key - Instructions for Use				
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93944	004/07.2022	Kornwebel, Olga 2022-07-05	Kornwebel, Michael 2022-07-05	Scherg, Tobias 2022-07-05

Safety Instructions and Technical Description

Licensing System

BESA products are protected by the Sentinel HL licensing system as developed by Thales.

The licensing system requires a USB key – that is, the BESA License Key 2.0 – and the device drivers to access the USB key.

If you selected the Install BESA License Key drivers option at the final stage of the installation, the device drivers for the BESA License Key 2.0 have been installed automatically on your computer. In order for BESA products to be run, a BESA License Key with valid license information must be attached to a USB port on a computer in your local area network. If you have not yet received it, you can obtain the BESA License Key from our sales department by contacting sales@besa.de.

If you have attached the BESA License Key 2.0 (USB key) directly to your computer or server and are not able to start the relevant BESA product, please refer to this document for troubleshooting tips.

System Requirements

The BESA License Key 2.0 is an essential accessory product to BESA products, and shall either be subject to constant supervision, or shall be connected to machines located in areas with access restriction. Failure of the IT-network and/or operating system on which the BESA License Key 2.0 is used, or failure of the BESA License Key 2.0 in general, may render BESA products which use the BESA License Key 2.0 as an accessory as unusable.

- For the purposes of communication between BESA products and the BESA License Key 2.0, the manufacturer uses TCP and UDP ports 1947. Please make sure to open these ports within your network or IT environment in order to enable the required communication. Once the license key runtime environment is installed, you can find more information on settings, security and general topics concerning the BESA License Key 2.0 under the following link:

http://localhost:1947/int/ACC_help_index.html

- HASPUUserSetup.exe installs the runtime environment required by the BESA License Key 2.0. The supplied runtime environment is provided by BESA “as is”, and according to the manufacturer is compatible with computers running the following operating systems or hardware:

(NOTE: Information supplied by manufacturer (Thales),

Source: https://docs.sentinel.thalesgroup.com/ldk/LDKdocs/RN/ReleaseNotes/130-Supported_Platforms_for_Sentinel.htm#Sentinel4,

downloaded on 2021-09-07, available in S:\External Programs\HASP (SafeNet, Sentinel, Gemalto)\Sentinel_LDK_Run-time_setup V. 8.21\ Supported Platforms for Sentinel LDK End Users and Vendors.pdf)

Windows

- Windows 7 SP1
- Windows 8.1 SP1
- Windows Server 2008 R2 SP1
- Windows Server 2012 R2
- Windows Server 2016
- Windows Server 2019
- Windows 10 IoT Enterprise 2019 LTSC
- Windows 10 Version 20H2

Note: Windows 10 Insider Preview builds are not supported.

The latest service packs and security updates must be installed.

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA License Key - Instructions for Use			 BESA®	
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93944	004/07.2022	Kornwebel, Olga 2022-07-05	Kornwebel, Michael 2022-07-05	Scherg, Tobias 2022-07-05

Virtual Machines The VM detection and VM fingerprinting capabilities provided by Sentinel LDK have been validated on the following technologies:

VMware Workstation 15
VMware ESXi 6.5, 6.7
Microsoft Azure

- The BESA License Key 2.0 is designed to be attached to a USB 2.0 or USB 3.0 port. Please note that you may need to use an adapter to attach the BESA License Key 2.0 to your PC (the adapter is not included).
- The BESA License Key 2.0 has been tested by BESA on the following operating systems:
 - Windows 10 Professional 64bit version 2004
 - Windows 10 Professional 64bit version 20H2
 - Windows 7 Professional 64bit Service Pack 1

Please note, that changes of the hardware and software platforms (e.g., with patches/updates of antivirus/firewall software, system libraries, firmware, and others) and/or changes to the IT network may result in the possibility of safety and/or security vulnerabilities, and/or reduced functionality of BESA License Key 2.0. It is recommended to use the system combinations as defined within this document.

Please note, that the use of BESA License Key 2.0 on an IT-network could result in previously unidentified risks to patients, users or third parties, and that the user is responsible to identify, analyze, evaluate, and control these risks. Any subsequent changes to the IT-network could introduce new risks and require additional analysis.

Data Protection

BESA License Key 2.0 does not store any data.

Use in combination with other products

The license key is allowed to be used as an accessory product for the following BESA products:

- BESA Research (7.X; incl. all sub-versions)
- BESA MRI (3.X; incl. all sub-versions)
- BESA Connectivity (1.X; incl. all sub-versions)

The license key is allowed to be used with the following support tools of BESA:

- Diagnostics.exe (8.X; incl. all sub-versions)
- License Search.exe (3.X; incl. all sub-versions)
- HLUpdate.exe (1.X; incl. all sub-versions)

BESA releases are constantly checked with other installations (e.g. MS Office etc.) on the same hardware or software platforms as defined in section System Requirements.

If any incompatibilities are observed, they are documented within the Update History of the product. For further questions and information please use the channels provided on www.besa.de.

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA License Key - Instructions for Use			 BESA®	
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93944	004/07.2022	Kornwebel, Olga 2022-07-05	Kornwebel, Michael 2022-07-05	Scherg, Tobias 2022-07-05

Application Environment

Please note: It is the User's responsibility to ensure the safety of combined medical or medical-&-non-medical-devices particularly installed in the patient environment (whether in institutes, clinics or practitioners' location).

The user of our product has to ensure that such combination fulfil the same safety condition as a single medical device according to IEC 60601-1.

Such combination with any non-medical device establishes are "in-house medical system" – observe the latest Medical Device Operator Ordinance (e.g. MedBetreiberVerordnung, Germany 2020) or your national Operator Laws.

We recommend you to document / retrace our Software version/updates installed in your facility / network in a MPG-Medizingeräte-Buch (medical facilities book), similar to those needed for Hardware.

This BESA product does not affect the possibility of using any medical computer within patient environment.

BESA GmbH is not liable for any use/installation beyond the defined PC network.

Installation and Un-Installation

The installation and un-installation of the product must be carried out by the system administrator or an authorized person. This staff bears full responsibility for correct setup of the product within the network.

License updates must also be carried out by the system administrator or an authorized person.

Please note, that in case of any problems during the installation, the HASPUsersetup.exe will trigger a rollback of current changes. In that case, please figure out and eliminate the issue, and run the HASPUsersetup.exe again.

Installing Sentinel Runtime

If you need a separate setup file for HASPUserSetup.exe to install the driver on a server to which the BESA License Key 2.0 is attached but no BESA products are running, for example, you can download a current version of HASPUserSetup.exe that has been released for use with BESA products:

<https://www.besa.de/downloads/besa-license-key/>

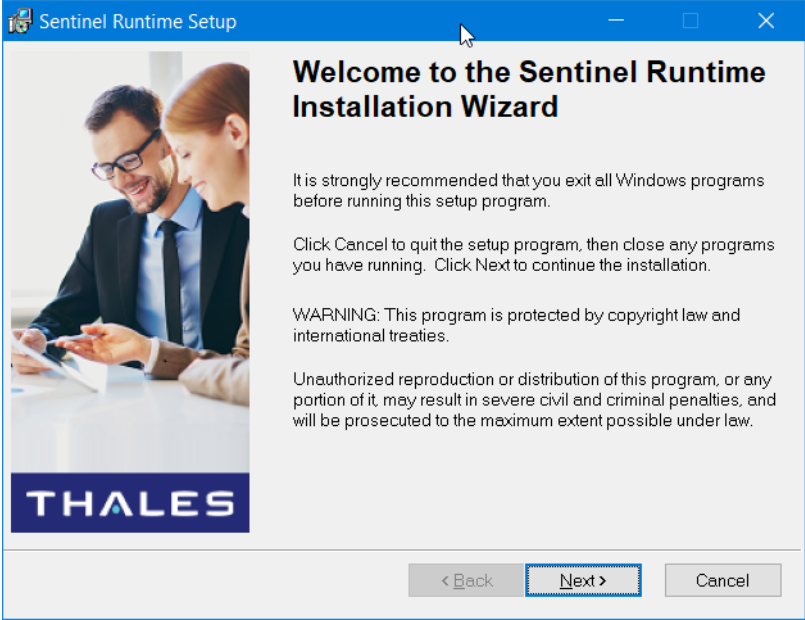
If you apply updates to HASPUserSetup.exe, make sure to roll out the updates to all machines running BESA products, or to update the machine to which the BESA License Key 2.0 is physically attached.

To install the software required to use the BESA License Key 2.0, you need to have administrator privileges on your computer. The screenshots below are taken from an installation on a computer running Windows 10. They may look slightly different on other operating systems.


Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA License Key - Instructions for Use				
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93944	004/07.2022	Kornwebel, Olga 2022-07-05	Kornwebel, Michael 2022-07-05	Scherg, Tobias 2022-07-05

1. The Sentinel Runtime Installation Wizard opens. Press the Next button to continue.

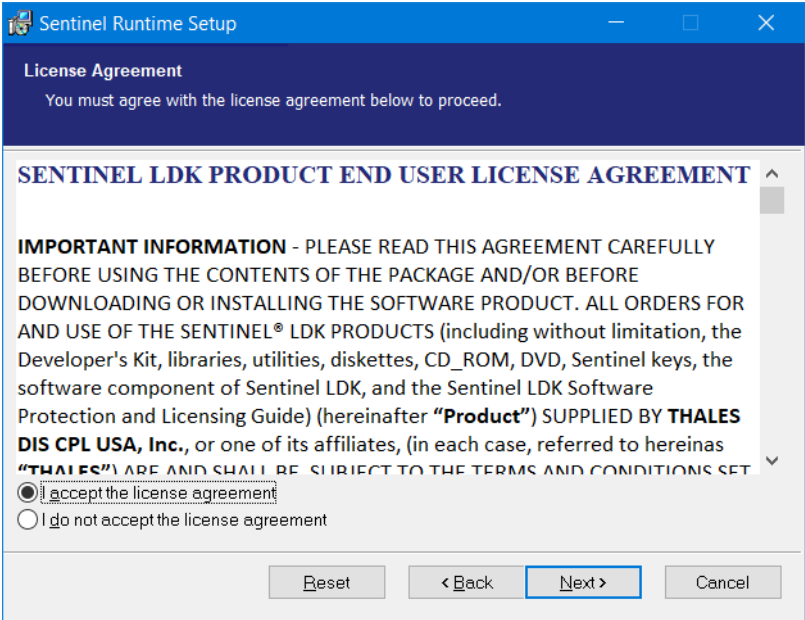
2. Please read the license agreement. If you agree to the terms, select I accept the license agreement and press the Next button to continue.


Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA License Key - Instructions for Use			 BESA®	
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93944	004/07.2022	Kornwebel, Olga 2022-07-05	Kornwebel, Michael 2022-07-05	Scherg, Tobias 2022-07-05

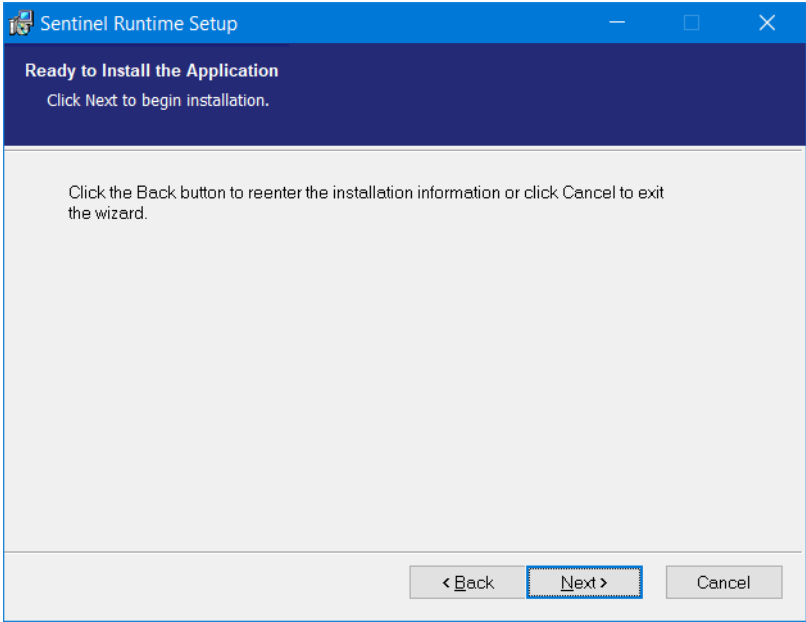3. The runtime is ready for installation. Please press theNext button to continue

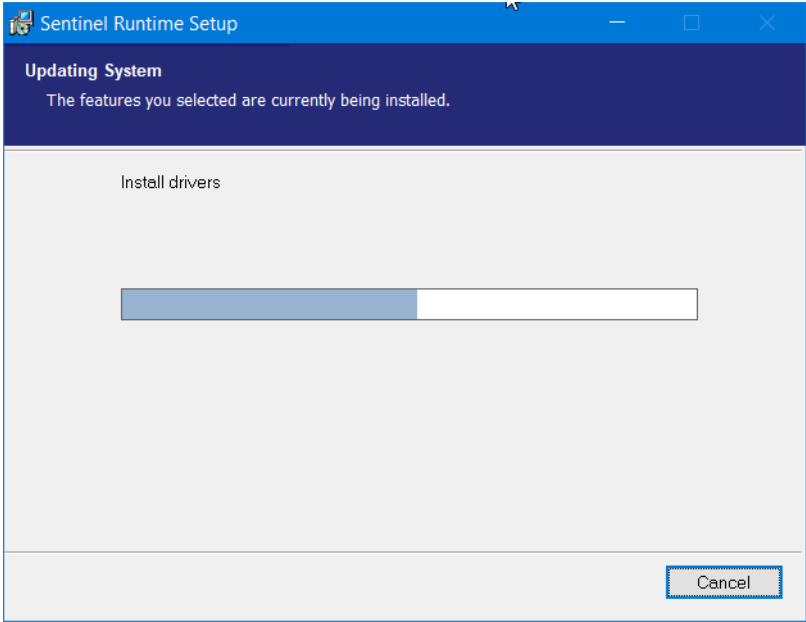
4. It may take a few minutes to install the drivers and updatethe system.

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA License Key - Instructions for Use				
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93944	004/07.2022	Kornwebel, Olga 2022-07-05	Kornwebel, Michael 2022-07-05	Scherg, Tobias 2022-07-05

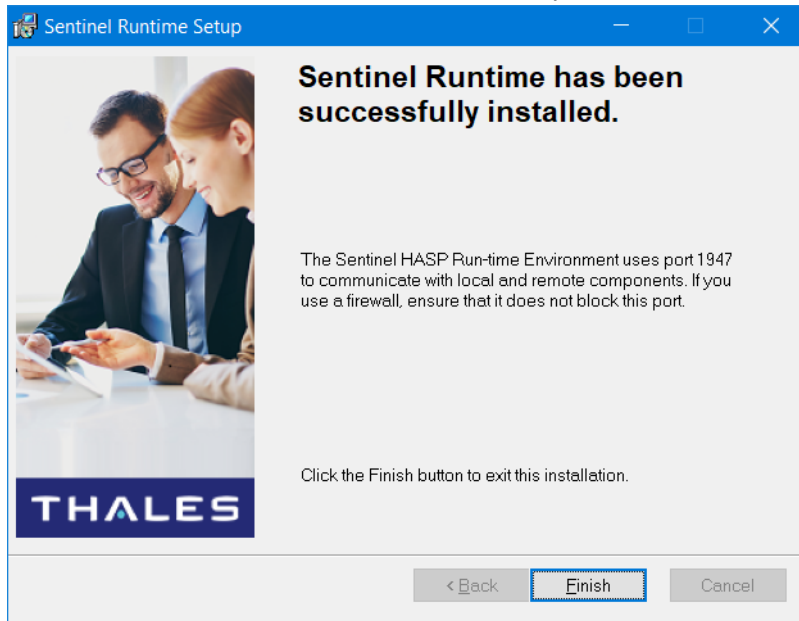- Installation of the Sentinel Runtime is now complete. Press the Finish button.

Uninstalling Sentinel Runtime

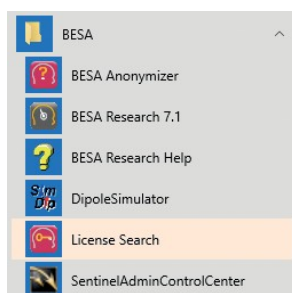
Uninstallation of the product must be carried out by the system administrator or an authorized person. This person bears full responsibility for correct deinstallation of the product within the network.

In the Windows operating system, open “Control Panel\All Control Panel Items\Programs and Features”. Search for “Sentinel Runtime”, click on the name and select Uninstall.

The uninstallation wizard will run, uninstalling the HSAPUsersetup.exe (Sentinel Runtime).

License Search

- To check if your BESA License Key 2.0 contains a valid license for starting the relevant BESA product, start the License Search application using the shortcut in the BESA program group in the start menu.


Alternatively, press the Start button and enter “License Search” in order to find the application. You will see the License Search application in the search results. Open it.

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA License Key - Instructions for Use			 BESA®	
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93944	004/07.2022	Kornwebel, Olga 2022-07-05	Kornwebel, Michael 2022-07-05	Scherg, Tobias 2022-07-05

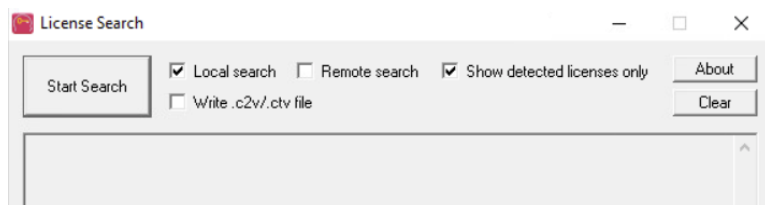- If the BESA License Key 2.0 is connected locally, select the Local Search option and press the Start Search button.

If you connected the key to a server or to another machine in your network, select the Remote Search option instead. If the dongle is displayed, please check the license state and start the relevant BESA product. If the dongle is not displayed, please see the next pages for a solution to the problem.

If installation of the BESA License Key 2.0 fails after carrying out the above steps, the most common reason is that the automatic installation of the device drivers has failed. Please proceed as follows:

- Disconnect the BESA License Key 2.0 from the local personal computer.
- Navigate to the C:\Program Files (x86)\BESA\<Your BESA product>\Utilities\License\System\ folder located on the hard disk, and run the HASPUserSetup.exe to install the device drivers manually.

Troubleshooting

If BESA License Key 2.0 is still not working correctly, it may be due to one or more of the following reasons:

- BESA License Key 2.0 is not connected.

Solution: Ensure that BESA License Key 2.0 is accessible in order to run the program. If BESA License Key 2.0 is installed correctly, the Windows Device Manager lists three items in the “Universal Serial Bus controllers” section:

- Sentinel HASP Key
- Sentinel HL Key
- Sentinel USB Key

- USB port is defective.

Solution: Check if the LED light is illuminated on the BESA License Key 2.0. If not, remove and re-insert the key. Wait for the operating system to detect the device. If it still fails to light up, try another USB port or request a replacement dongle.

- Dongle is faulty.

Solution: On rare occasions, the BESA License Key 2.0 may be faulty. The dongle does not light up in this case and is not detected in the Device Manager. Request a replacement dongle.

- Sentinel LDK License Manager Service is blocked by a fire- wall or an antivirus application.

Solution: Add C:\Windows\system32\hasplms.exe to the list of the antivirus and firewall exceptions.

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA License Key - Instructions for Use				
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93944	004/07.2022	Kornwebel, Olga 2022-07-05	Kornwebel, Michael 2022-07-05	Scherg, Tobias 2022-07-05

- Port 1947 is blocked by a firewall application.
Solution: Add port 1947 to the list of firewall exceptions.
- Sentinel LDK License Manager Service is stopped.
Solution: Restart the Sentinel LDK License Manager Service, available in (Control Panel → Administrative Tools → Services).

Individual License Settings: Sentinel Admin Control Center (ACC)

The Sentinel runtime environment includes the device drivers and the Sentinel Admin Control Center (ACC). ACC can be used to monitor licenses and sessions when a protected application is operating, and to manage detachable licenses. To access the ACC, please start the Sentinel Admin Control Center program from the Apps/ BESA program folder, or simply enter the following address in your internet browser (an internet connection is not required): <http://localhost:1947>

Sentinel Admin Control Center Help

Sentinel Keys Host Name: MachineName

Sentinel Keys

Products

Features

Sessions

Update/Attach

Access Log

Configuration

Diagnostics

Location	Vendor	Key ID	Key Type	Configuration	Version	Sessions	Actions
Local	YWCXE (87128)		Reserved for New SL Key	SL	8.11		Fingerprint
Local	YWCXE (87128)	123456789	Sentinel HL NetTime 10	HASP	4.34		Products Features Sessions Blink on
server123	YWCXE (87128)	987654321	Sentinel HL NetTime 10	HASP	4.31	6	Browse Net Features

Important administrative options:

The Sentinel Keys page shows which BESA License Keys are currently present in the network, including locally connected keys. The Sessions page lists all client sessions on the local machine, and also client sessions logged in remotely to the local machine. This page enables you to view session data and to delete sessions.

The Configuration page enables you to specify basic settings. You can also configure access to or from other computers. To determine that the license on the local BESA License Key can be used only locally on that particular computer, for example, switch to the Access from Remote Clients tab and disable the Allow Access from Remote Clients option.

Note: that changes to Configuration page may result in restriction of availability of licenses for BESA products, and thus provide these products from running. Access to machines where ACC is installed shall therefore be restricted.

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA License Key - Instructions for Use			 BESA®	
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93944	004/07.2022	Kornwebel, Olga 2022-07-05	Kornwebel, Michael 2022-07-05	Scherg, Tobias 2022-07-05

Exclusions for Use

N/A

Contraindication to patients

N/A

BESA is not liable for the use outside of the rules defined in the instructions for use documents.

Manufacture, Transport, Storage and Usage conditions

Environmental conditions for manufacturing

N/A

Environmental conditions for Transport, Storage and Usage

During extended periods of transport, storage and usage please observe the following conditions:

Temperature: - 20°C to 60°C

Humidity: 0% to 95%

Disposal information

The BESA License Key 2.0 and the box in which it is delivered must be disposed of in compliance with national guidelines on environmental protection.

Countries and Markets

This BESA product may be sold in the following countries and markets:

Austria	Germany	Malta	Spain	China	Russia
Belgium	Greece	Macedonia	Sweden	Colombia	Saudi Arabia
Bulgaria	Hungary	Netherlands	Switzerland	India	Singapore
Croatia	Iceland	Norway	Turkey	Iran	South Africa
Cyprus	Ireland	Poland	United Kingdom	Israel	South Korea
Czech Republic	Italy	Portugal	Australia	Japan	Taiwan
Denmark	Latvia	Romania	Argentina	Malaysia	Thailand
Estonia	Liechtenstein	Serbia	Brazil	Mexico	Turkmenistan
Finland	Lithuania	Slovakia	Canada	New Zealand	UAE
France	Luxembourg	Slovenia	Chile	Qatar	USA

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

BESA License Key - Instructions for Use				
Document ID	Revision Number	Edited by	Reviewed by:	Released by / Date
93944	004/07.2022	Kornwebel, Olga 2022-07-05	Kornwebel, Michael 2022-07-05	Scherg, Tobias 2022-07-05

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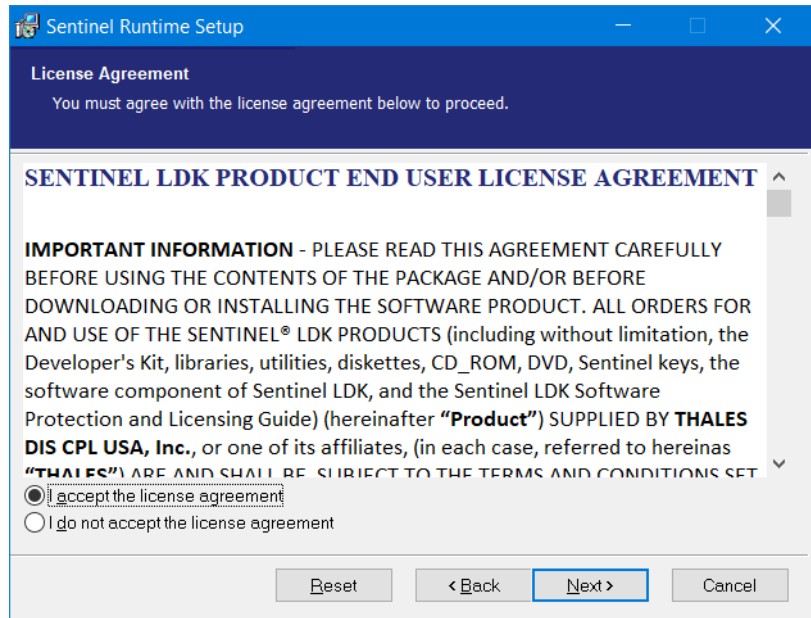
License Agreement
You must agree with the license agreement below to proceed.

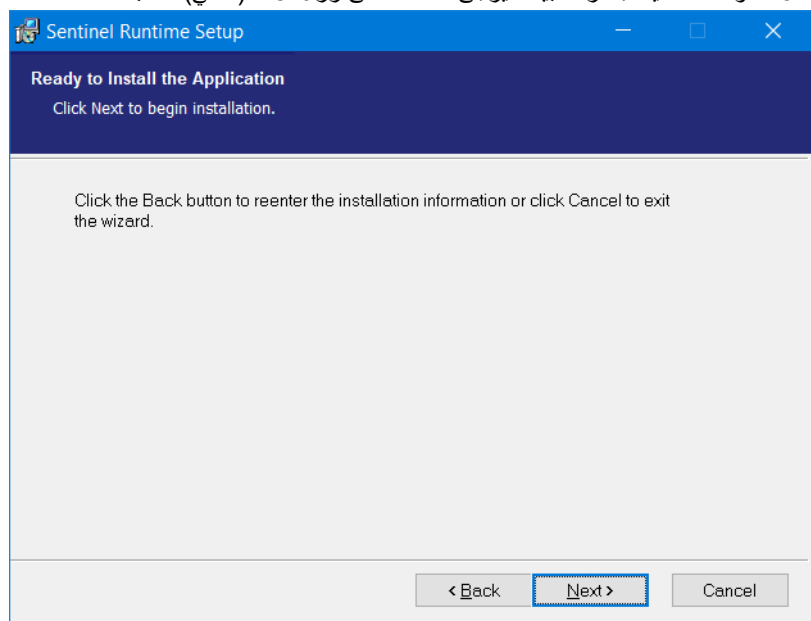
SENTINEL LDK PRODUCT END USER LICENSE AGREEMENT

IMPORTANT INFORMATION - PLEASE READ THIS AGREEMENT CAREFULLY BEFORE USING THE CONTENTS OF THE PACKAGE AND/OR BEFORE DOWNLOADING OR INSTALLING THE SOFTWARE PRODUCT. ALL ORDERS FOR AND USE OF THE SENTINEL® LDK PRODUCTS (including without limitation, the Developer's Kit, libraries, utilities, diskettes, CD_ROM, DVD, Sentinel keys, the software component of Sentinel LDK, and the Sentinel LDK Software Protection and Licensing Guide) (hereinafter "**Product**") SUPPLIED BY **THALES DIS CPL USA, Inc.**, or one of its affiliates, (in each case, referred to hereinafter as "**THALES**") ARE AND SHALL BE SUBJECT TO THE TERMS AND CONDITIONS SET FORTH IN THE SENTINEL LDK SOFTWARE PROTECTION AND LICENSING GUIDE.

☒ I accept the license agreement
☐ I do not accept the license agreement

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Ready to Install the Application
Click Next to begin installation.

Click the Back button to reenter the installation information or click Cancel to exit the wizard.

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来源: [https://docs.sentinel.thalesgroup.com/ldk/LDKdocs/RN/ReleaseNotes/130-](https://docs.sentinel.thalesgroup.com/ldk/LDKdocs/RN/ReleaseNotes/130-Supported_Platforms_for_Sentinel.htm#Sentinel4)

[Supported_Platforms_for_Sentinel.htm#Sentinel4](https://docs.sentinel.thalesgroup.com/ldk/LDKdocs/RN/ReleaseNotes/130-Supported_Platforms_for_Sentinel.htm#Sentinel4), 下载于2021-09-07, 可在以下位置获取 S:\External Programs\HASP (SafeNet, Sentinel, Gemalto)\Sentinel_LDK_Run-time_setup V. 8.21\ Supported Platforms for Sentinel LDK End Users and Vendors.pdf)

Windows Windows 7 SP1
 Windows 8.1 SP1
 Windows Server 2008 R2 SP1
 Windows Server 2012 R2
 Windows Server 2016
 Windows Server 2019
 Windows 10 IoT Enterprise 2019 LTSC

Template document name: T78 Instructions for Use	Template edited on/by: 2022-07 / TO, MK
Template valid from: 2022-07-05	Template released on/by: 2022-07 / MK
Template Document ID: 93917	

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