

BESA License Key 2.0 - External Release Notes

Documents are subject to workflows, and content shall only be used if the content is approved (indicated by a green ring-symbol)

Approved , Changed on: January 22nd 2024 at 1:47:04 PM (UTC)



External Release Notes

Table of Contents

- [External Release Notes](#)
 - [BESA License Key Release Notes](#)
 - [Version 2.0.1.1](#)
 - [HASPUserSetup.exe \(V. 9.14\):](#)
 - [Previous Versions](#)
 - [Version 2.0.1.0](#)
 - [HASPUserSetup.exe \(V. 9.12\):](#)
 - [Version 2.0.0.0](#)
 - [Sentinel HL NetTime 10 \(V. 6.08\)](#)
 - [HASPUserSetup.exe \(V. 8.21\):](#)
 - [Page ID](#)
 - [Change History \(valid revision number\)](#)
 - [Attachments](#)
 - [Product Change Notification- Sentinel HL](#)
-

BESA License Key Release Notes

Version 2.0.1.1

HASPUserSetup.exe (V. 9.14):

Info

HASPUserSetup.exe is a GUI-based installation program to install the run-time environment for the BESA License Keys.

This utility is supplied by BESA license key manufacturer Thales and is provided by BESA as is.

The utility and the corresponding readme file are located in \Utilities\License\System folder of the installation directory of BESA programs.

Improvements / Bugfixes / Residual anomalies

- The Diagnostics page in Admin Control Center displays the operating system correctly on a Windows 11 machine. [SM-143338]
- Under certain circumstances, UDP packets may be lost. Broadcast search now repeats the UDP transmission multiple times to overcome the loss of UDP packets. [SM-134020]
- Previously, when a proxy was configured in ACC, the LM would attempt to resolve DNS names, even if they were not resolvable in the local network. This resulted in connection failures. Now, when a proxy is defined in ACC, the LM no longer attempts to resolve DNS names. Instead, it allows the proxy to handle DNS resolution. [SM-139869]

For the complete list of known issues provided by Thales, please refer to the *Readme.html* file in \Utilities\License\System folder of the installation directory of BESA programs.

Previous Versions

Version 2.0.1.0

HASPUserSetup.exe (V. 9.12):

Improvements / Bugfixes / Residual anomalies

- When installing the RTE on a machine where Fabula Tech USB over Network is installed, installation of the RTE hangs.
Workaround: Disconnect the HL key before running the RTE installer.

Version 2.0.0.0

Sentinel HL NetTime 10 (V. 6.08)

Improvements

- **Updated versions V. 4.31/V. 4.34 to 6.08**
 - BESA License Keys were updated with an alternate smart card chip. These keys are fully backward compatible. To help visually identify the different key version: The silver keyring was replaced with a gold keyring. (Please refer to attached document *RoxtralD 92388 - Product Change Notification- Sentinel HL.pdf, Rev. 001/09.2021*)

Bugfixes

None

Residual anomalies

None

HASPUserSetup.exe (V. 8.21):

Improvements / Bugfixes / Residual anomalies

- Please refer to the *Readme.html* file in \Utilities\License\System folder of the installation directory of BESA programs.


Additional Known Issues (older versions)

Affected Tool + Version	Publication Date	Information
HASPUserSetup.exe (V. 7.100 and older)	November 2021	[KB0021742] Upgrade of Windows 10 to version 2004 may fail when working with run-time environment installed by HASPUserSetup.exe version 7.100 or earlier Please refer to the original manufacturer article for more information and workarounds: link to KB0021742

Change History (valid revision number)

Version	Effective Date	Approvals	Version Notes
1	Wed, Oct 11, 2023, 09:24:52 AM	>> *Action: approve, MK, Approval: QM Approval (State: QM Approval, Wed, Oct 11, 2023, 09:24:52 AM)	None
2	Wed, Oct 11, 2023, 12:06:02 PM	>> *Action: approve, MK, Approval: QM Approval (State: QM Approval, Wed, Oct 11, 2023, 12:06:02 PM)	None
3	Mon, Jan 22, 2024, 13:47:04 PM	>> *Action: approve, MK, Approval: QM Approval (State: QM Approval, Mon, Jan 22, 2024, 13:47:04 PM)	None

Attachments

Attachment	Description
Product Change Notification- Sentinel HL  RoxtraID 92388 - Product Change Notification - Sentinel HL.pdf	Product Change Notification- Sentinel HL

▼ Template Change History

TEMPLATE	TEMPLATE Version	Effective Date	Version Notes
External Release Notes Template	1	Mon, 13 Mar 2023 15:57:29 GMT	None
External Release Notes Template	2	Tue, 21 Mar 2023 14:18:43 GMT	None
External Release Notes Template	3	Tue, 02 May 2023 08:13:02 GMT	None
External Release Notes Template	4	Thu, 25 May 2023 06:11:38 GMT	None
External Release Notes Template	5	Fri, 23 Jun 2023 13:56:29 GMT	None